

MAXCRUSH™ M200 Cup Pill Crusher

Instructions for Use

MAXCRUSH™ M200 Cup Pill Crusher and Plastic Crusher Cups are an effective system to crush tablets into easy to consume fine powder.

An ergonomic and innovative way to CRUSH AND GRIND pills into easy to consume fine powder.

Instructions for Use



1. Place pill(s) into a crusher cup and then cover with a second crusher cup. Slide the cups on the handle anvil. We recommend 1 oz. (30 mL) plastic cups.



2. Gently lower the handle until the two anvils meet. This is the starting point. Keep your wrists in a straight or neutral posture. Use two hands for even less effort.



3. Apply gentle pressure to the handle. This alone will crush the pills. You do not need to pound or hammer!



4. Between each crush, release the handle to the start position (you do not need to lift the handle up) and gently turn the cups. The pills will crush with the spring/twisting motion of the anvils.



5. Your result is a fine powder with minimal effort.

MAXPERT
Medical

Expert Products for Expert Care

Cleaning and Maintenance

The only maintenance required for the MAXCRUSH™ M200 Cup Pill Crusher is the occasional wipe with a damp cloth.

Limited Warranty

Manufacturer warrants that the MAXCRUSH™ Cup Pill Crusher is free of parts defect due to workmanship and functional failure for two years from date of purchase. We recommend the use of 1 oz. (30 mL) plastic cups for crushing.

Call: 866-231-1222
www.maxpertmedical.com