



MAXSPLIT™ PILL SPLITTER WITH PUNCHER INSTRUCTIONS FOR USE

OPTIONAL: The Pill Puncher feature can be used to aid in removing pills from medication packaging.

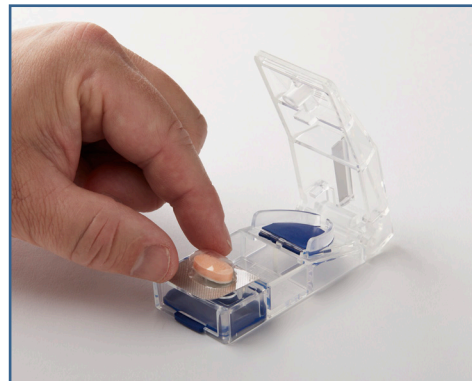

1. With the splitter on a solid, flat surface, open splitter and place blister pack pill side up, centered over the keyhole.

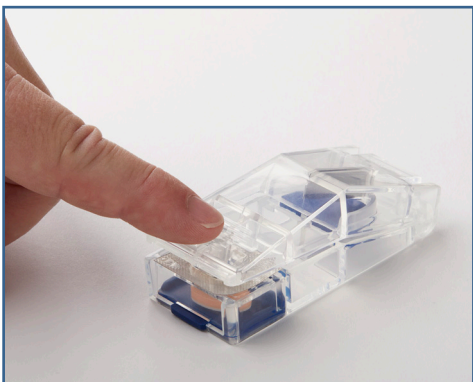

2. Close and press the lid with slight pressure to push pill through blister pack. The pill will be punched out of the blister and through keyhole into a compartment. Remove the empty blister package.



3. Lift the splitter, open the compartment cover and there you will find your pill. Continue to splitting steps on page 2 for splitting pill.

Note: The puncher will work with many types of packing but not all. The type of package and pill size and shape can greatly affect the puncher function. Some packages may require additional pressure on the splitter lid and require good alignment.





CLEANING: Wash in warm water being careful not to directly contact the sharp blade. Rinse and allow to air dry prior to using.

MAXPERT®
Medical
Expert Products for Expert Care


MAXSPLIT™ PILL SPLITTER WITH PUNCHER INSTRUCTIONS FOR USE

SPLITTING PILLS



WARNING: Avoid direct contact with the sharp blade. Keep splitter and medication out of reach of children. Consult your physician or pharmacist prior to splitting pills.

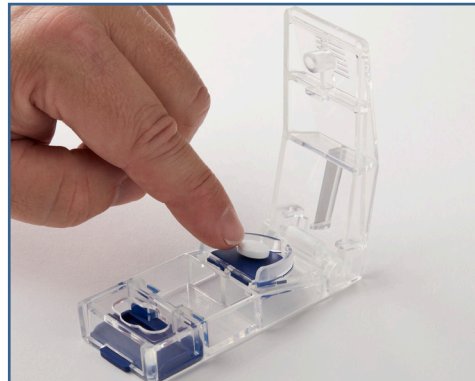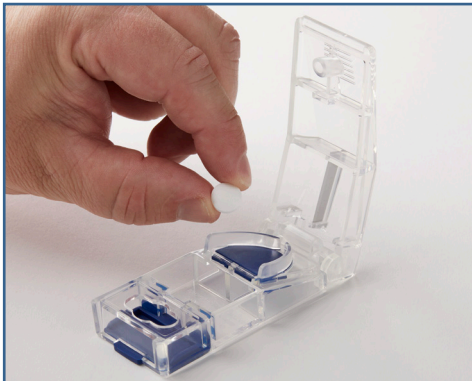

1. With the splitter on a flat solid surface, open the lid and insert a pill into the U-shaped holder. The holder can hold round and oblong pills of various sizes. The rubber insert lining the U-shaped area will help keep pill in place during splitting.

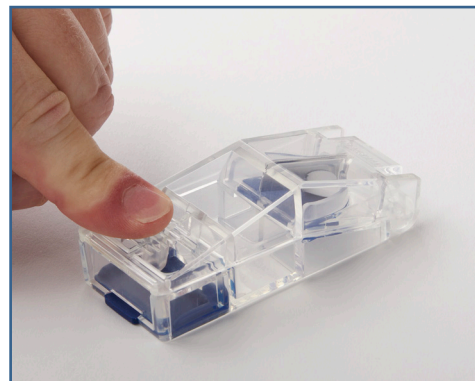

2. Once your pill is in position, close and press lid to split pill.

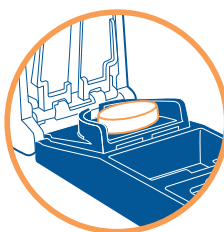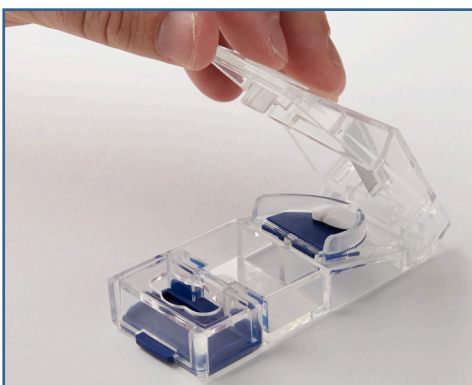

Split various shapes and sizes of pills.



3. Open and remove split pill, leaving any remaining in the storage compartment. Should you need to split pill again, repeat steps 1-3.

MAXPERT®
Medical

Expert Products for Expert Care

CLEANING: Wash in warm water being careful not to directly contact the sharp blade. Rinse and allow to air dry prior to using.