


SILENT KNIGHT® Pill Crusher In-Service

The Silent Knight is a simple, safe and efficient way to crush pills from solid to fine powder. When following the below steps and procedure it is possible to crush a single pill or multiple pills quickly and effectively.

IN-SERVICE STEPS:

1. Place pill(s) into Silent Knight Pill Crusher Pouch.
Lift handle and place pouch down between the crusher plates to initiate crushing.
2. Press handle down to crush pill(s). Lift handle and pull pouch upward to the middle of the crusher plates.
3. Then press the handle down to activate grinding movement.
Press handle down several times to grind pill(s) into fine powder. Repeat if needed.
4. Remove the pouch and add the medication to an appropriate serving medium for administration.

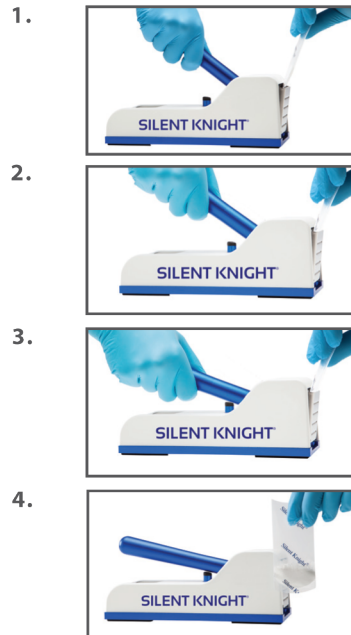

Distributed by:



SILENT KNIGHT EXCLUSIVE PILL CRUSHER POUCH

The Silent Knight does not function alone. Its companion item, the patented Silent Knight Pill Crusher Pouch, is essential to guarantee maximum performance while reducing cost, waste, malfunction and lost time.

The pouch's distinctive curved bottom structure assures full recovery of crushed medication. The thick, durable construction effectively reduces pharmaceutical cross contamination and eliminates loss of medication.

Note: No other pill crusher pouch will effectively and safely function in the Silent Knight. Use of any other pill pouch will compromise product performance and outcome.

LIFETIME WARRANTY

The Silent Knight is guaranteed for life to be free of materials and parts defects. If the unit fails due to parts failure it will be replaced at no charge to the original owner. NOTE: The use of any pill container other than the Silent Knight Pill Crusher Pouch (i.e. other plastic pouches, plastic unit dose packages, or cups) will void the warranty.

SPECIFICATIONS & ORDERING INFORMATION

Silent Knight Pill Crusher - Materials: Stainless steel, anodized aluminum. Latex free.

Silent Knight Pill Crusher Pouch - Materials: Low Density Polyethylene (LDPE).

Silent Knight Pouch is FDA approved for pharmaceutical packaging.

MANUFACTURER'S CLEANING & MAINTENANCE RECOMMENDATION

The Silent Knight is made entirely of non-rusting materials. When dirty it may be cleaned with soap and water and a damp cloth. Do not use bleach. If your facility follows a disinfection protocol, please note that there are commercially available germicidal wipes that are EPA registered and compliant with the CDC guideline for disinfection and sterilization in healthcare facilities. Two examples of acceptable germicidal wipes are from Professional Disposables Inc. (PDI) - Super Sani-Cloth Germicidal Wipe and Sani-Cloth AF3 Germicidal Wipe. Any use of bleach or a bleach based disinfectant wipe will cause discoloration and void the lifetime warranty.

Silent Knight® is a registered trademark of Links Medical Products, Inc.
Protected by U.S. patents 7,051,963 B1, D497, 543 5,5, 915,637, 9 125, 796