

# Maxpert® Heavy Duty Fold Over Tear Top Pill Crusher Pouch Instructions for Use - Item BCB2805



1). Open the pouch. Slight squeeze helps.



2). Insert pill(s).



3). Fold the top where it reads fold here to close. (Insert into crusher and crush medications while keeping folded)



4). Once medication is crushed you can empty two ways. Reopen the top and pour along the folded straight side of pouch.



5.) The second option to empty crushed medications is to remove the top of the pouch at the perforation. You'll see a small notch to tear.



6.) Once the top is removed, pour along the folded straight side of the pouch. This side traps less medication when emptying.