

SILENT KNIGHT®

ProShield HD **SK800**

Pill Crusher In-Service

The Silent Knight ProShield HD SK800 is a simple, safe and efficient way to crush pills from solid to fine powder. When following the below steps and procedure it is possible to crush a single pill or multiple pills quickly and effectively.

IN-SERVICE STEPS:

1. Place pill(s) into ProShield HD Zip-Seal Pouch.
Lift handle and place ProShield HD Zip-Seal Pouch down between the crusher plates to initiate crushing.
2. Press handle down to crush pill(s). Lift handle and pull ProShield HD Zip-Seal Pouch upward to the middle of the crusher plates.
3. Then press the handle down to activate grinding movement. Press handle down several times to grind pill(s) into fine powder. Repeat if needed.
4. Remove the ProShield HD Zip-Seal Pouch and add the medication to an appropriate serving medium for administration.



Distributed by:



CLEANING & MAINTENANCE

The **Silent Knight ProShield HD SK800 Pill Crusher** is made entirely of non-rusting materials and may be cleaned regularly with a damp cloth. A facility approved disinfectant wipe may also be used when indicated. Always follow your facilities cleaning and disinfecting protocols.

SKP31126